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Request form for DNA typing of dogs for parentage control

Hundägare ifylles av ägaren / to be completed by the owner

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Skicka provet till SLU snarast möjligt efter provtagning. *Send the sample to SLU as soon as possible after sampling.*

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Sveriges lantbruksuniversitet
Swedish University of Agricultural Sciences

Animal Genetics Laboratory

Information to the dog owner and sampling veterinarian

Instructions for sampling

Blood

A blood sample (1-3 ml) is taken by a veterinarian in an EDTA-tube. This tube should be clearly labeled with the dog's name, registration number or ID number. Please invert the tube 2-3 times immediately after sampling to avoid coagulation. The sample should be sent the same day, or refrigerated prior to sending at a later time. Samples can be shipped at room temperature.

Buccal Swabs

There are two swabs for each individual to be sampled (**ordered from the Animal Genetics Laboratory**). One for each cheek of the individual to be sampled. The dog or puppy being sampled should not have suckled, or have eaten, in the 30-60 minutes prior to sampling. If possible, keep puppies separated from each other, and their mother, before sampling.

Open the package and remove the swab making sure not to touch bristles. To collect buccal cells, run the swab on one side of the cheek inside the mouth. Roll the swab around 10-20 times over a large area and with some pressure. Repeat this procedure with the second swab on the dog's *other* cheek. Allow the swabs to air dry for approximately 10-15 minutes at room temperature. Place the dried swabs back in the envelope. Mark the envelope with the dog's ID (name, chip number or barcode). The samples should be sent the same day, or refrigerated prior to sending at a later time. Samples can be shipped at room temperature.

Please note! The amount of DNA we can obtain from mouth swabs are often less, and sometimes of lesser quality than that obtained from blood. This means that we may have to ask for new swabs from your dog if the swab DNA amount is insufficient for the genetic test.

Payment

Payment according to the list price is made in advance. In the message, enter the reference "HBIO" and the dog's name. Name of account holder: Sveriges Lantbruksuniversitet, Account number (IBAN): SE92 1200 0000 0128 1011 9104, BIC/Swift: DABASESX, Bank address: Norrmalmstorg 1,111 46 Stockholm, Sweden.

The normal turnaround time for the test is approximately 10-15 days after the sample's arrival. The results are sent by email to the owner or stated recipient.

Note! Payment should be made prior to any service rendered and goods delivered.

Send the samples to the address below: Animal Genetics Laboratory Swedish University of Agricultural Sciences Box 7023 SE-750 07 Uppsala Sweden	Using a courier*, please use the following address: Animal Genetics Laboratory Swedish University of Agricultural Sciences VHC Godsmottagning Almas allé 4C SE-756 51 Uppsala Sweden
--	--

* Please contact the Animal Genetics Laboratory (hgenlab@slu.se) for an import permission, buccal swabs or for further questions.